

(continued) Appendix A. Operator PMCS

4110-01-352-3653

Refrigerator, Mechanical, Blood Bank, Model FT2TRBLB

[B-Before Operation, D-During Operation, A-After Operation, W-weekly, Q-Quarterly, and S-Semiannually]

ITEM NO	INTERVAL	ITEM TO BE INSPECTED AND PROCEDURE	IS NOT MISSION CAPABLE IF:
1	B	<p><b>Refrigerator</b></p> <p>a. Ensure that a copy of the manufacturer's manual is on hand.</p> <p>b. Inspect components for damage, discoloration, or excessively worn components.</p>	<p>The power cord is cracked or frayed or wires are not covered by the cord insulation.</p>
2	B  B  W	<p><b>Operating and Maintaining the Refrigerator</b></p> <p>a. Ensure that a three-prong grounding plug is being used in an appropriate electrical outlet as directed by the manufacturer's literature.</p> <p>b. Ensure that the interior of the refrigerator has reached the set temperature and that the compressor has cycled at least three times before loading product into the cabinet.</p> <p>c. Clean the refrigerator as directed by the manufacturer's literature. Be sure the interior of the cabinet is cleaned with mild soap and rinsed with a warm baking soda solution. Dry thoroughly.</p> <p><b>NOTE: Failure to dry thoroughly may result in the formation of mildew.</b></p>	<p>Grounding prong is missing from plug.</p> <p>The compressor fails to cycle or the refrigerator does not reach the required temperature.</p>