

MRI, DT2000A Defibrillator, Tester, NSN 6625-00-433-9063

The following instructions for cleaning the defibrillator tester were provided by the Medical Maintenance Operations Division at Tracy, CA:

- Use a soft cloth, dampened in warm, soapy water. Wring out excess water to cleanse unit and paddle surfaces.
- Do not use any petroleum distillate-based products, solvents or harsh abrasives on these surfaces (i.e. carbon cleaner, acetone, or A-jax).
- The defibrillator paddle surfaces should be kept clean and free of conductive gel.

